



Juzo SoftCompress

Compression bandages & liners for edema therapy



Compression Bandage Liner

Benefits & Features



The vertically-stitched foam channels promote lymph drainage.

Using Juzo SoftCompress products can significantly improve lymphatic flow. Compared to the often used foam or synthetic padding, SoftCompress products are durable and comfortable. SoftCompress is made with 100% cotton covering so its perfect for sensitive skin.

Characteristics

- 100% cotton covering, great for sensitive skin
- Core made from foam
- Vertically-stitched foam channels
- Machine wash
- Saves time when applying bandaging
- Makes self-bandaging easier

How it works



Visible effect of the Juzo SoftCompress liner on a leg that has secondary lymphedema.

The comfortable liner can be worn under short-stretch bandages, wraps or compression garments, and provide additional compression, cushioning and an ideal pressure distribution.

Juzo SoftCompress liners are used as an effective treatment aid for edema. The vertically-stitched foam channels create differences in pressure, thereby promoting lymph drainage, while at the same time preventing soft tissue fibrosis.

The liners avoid any constriction by the edges of the short stretch bandages and protect the bends of the joints. These features and their effect are what makes Juzo SoftCompress so unique.

Indications

- lymphedema
- phlebology
- lipedema
- fibrosis

Contraindications

- arterial insufficiency or degeneration
- deep vein thrombosis
- untreated congestive heart failure
- localized or systemic infection
- untreated malign cancer
- absent or severely impaired sensation

If there are any open wounds in the region being treated, these wound areas must first be covered with a suitable wound dressing or stockinette before applying Juzo SoftCompress products.

SoftCompress Liners

The comfortable Juzo SoftCompress liners can be worn under short-stretch bandages, compression wraps or medical compression garments, and provide additional compression, cushioning and an ideal pressure distribution. The SoftCompress liners are used for fast and effective bandaging and make self-bandaging easy for the patient. They are available in uni sizes and standard sizes.



SoftCompress lower leg available in uni size. Includes 2 malleo pads.



6831 SoftCompress compression roll



6835 SoftCompress compression sheet 6836



SoftCompress liner available in standard sizes. Includes hand and elbow pads.



6822 SoftCompress liner arm in uni size. Includes decorative cover, a hand pad and an elbow pad.






6823 Standard and uni sizes can be cut according to the cutting marks to adjust length.



Uni size

SoftCompress liners in uni size have an open backside and can be fixed with elastic tapes or an elastic textile tube. Thereby the liner can be adjusted individually on highly changing limb circumferences.

Juzo SoftCompress Bandage Liners Leg


Description	Item #	Universal Size	Standard Size 1-2	Standard Size 1-4
 Bandage Liner Lower Leg incl. 2 malleo pads	6831	•		
 Bandage Liner, Roll size 1 – 5 mm, 8 cm x 4 m size 2 – 5 mm, 10 cm x 4 m size 3 – 5 mm, 12 cm x 4 m size 4 – 5 mm, 16 cm x 4 m	6835			•
 Bandage Liner, Sheet size 1 – 65 x 65 cm size 2 – 125 x 100 cm	6836		•	

Juzo SoftCompress Bandage Liners Arm

Description	Item #	Universal Size	Standard Size 1-6	
 Bandage Liner Arm incl. 1 hand pad 1 elbow joint pad	6822		•	
 Bandage Liner Arm incl. 1 hand pad 1 elbow joint pad	6823	•		

Size charts – Standard Sizes

Bandage arm · 6822

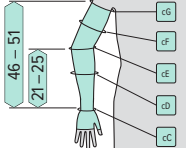Lengths in cm	Circumferences in cm	1 x-small	2 small	3 medium	4 large	5 x-large	6 xx-large
	cG	30 – 33	32 – 35	34 – 37	36 – 39.5	38 – 41.5	40 – 44
	cF	28 – 30.5	30 – 33	32 – 35	34 – 37	36 – 39.5	38 – 41.5
	cE	25 – 27.5	27 – 29.5	29 – 31.5	30.5 – 33.5	32 – 35	33.5 – 36.5
	cD	23 – 25	24.5 – 27	26 – 28.5	27.5 – 30	29 – 31.5	30.5 – 33.5
	cC	14.5 – 16	16 – 17.5	17.5 – 19	19 – 20.5	20.5 – 22	22 – 24

Accessories

Juzo SoftCompress Pads can be worn under compression bandages. They are especially suitable for a targeted increase in pressure for particularly localized edemas.




Breast Pad

6837

Juzo SoftCompress Breast Pad

The Juzo SoftCompress Breast Pad serves as a comfortable pad for a compression bra or chest bandage, e.g. for breast or chest wall edemas.

Description	Item #	Standard size
		1-3
 Breast pad size 1 – cup A-B size 2 – cup C-D size 3 – cup E-F	6837	•




Breast Bandage

6838

Juzo SoftCompress Breast Liner

The Juzo SoftCompress Breast Liner combines the positive characteristics of the Juzo SoftCompress material with those of a compression bra. It has an adjustable chest circumference. The breast liner can, for example, be used for breast or chest wall edemas.

Description	Item #	Standard size
		1-6
 Breast Liner size 1 – cup A size 2 – cup B size 3 – cup C size 4 – cup D size 5 – cup E size 6 – cup F	6838	•




Female Genital Pad

6839

Juzo SoftCompress Female Genital Pad

The Juzo SoftCompress Pad for female genital edema is worn underneath the compression garment. It provides a targeted increase in pressure, cushioning and an ideal pressure distribution when managing genital edema.


Description	Item #	Standard size
		1-3
 Female Genital Pad size 1 – 20 x 20 cm size 2 – 30 x 28 cm size 3 – 40 x 32 cm	6839	•



Genital pad men 6840

Juzo SoftCompress Genital Pad Men

The Juzo SoftCompress pad for male genital edema is worn underneath the compression garment. The broad channel structure is suitable for fibrotic tissue while the narrow structure additionally promotes lymph drainage. It provides a targeted increase in pressure and an ideal pressure distribution when treating genital edemas.

Description	Item #	Standard size
		1-3
 Genital Pad men size 1 – 28 x 20 cm size 2 – 30 x 23 cm size 3 – 32 x 26 cm	6840	•



Knee pad 6845

Juzo SoftCompress Knee Pad

The Juzo SoftCompress Knee pads cushion and soften the area in the bend of the knee joint.

Description	Item #	Universal Size
 Knee Pad 13 x 17 cm	6845	•



Elbow pad 6842

Juzo SoftCompress Elbow Pad

The Juzo SoftCompress Elbow pads cushion and soften the area in the bend of the elbow joint.


Description	Item #	Universal Size
 Elbow Pad 10 x 13 cm	6842	•



Malleo pad 6843

Juzo SoftCompress Malleo Pad

Juzo SoftCompress Malleo Pads cushion the Bisgaard (perimalleolar) region of the ankle. They provide a targeted increase in pressure and an ideal pressure distribution. The pads are delivered in pairs.

Description	Item #	Universal Size
 Malleo pad 6.5 x 9 cm delivered in pairs	6843	•



Hand pad 6844

Juzo SoftCompress Hand Pad

Juzo SoftCompress Hand Pads are especially for the back of the hand with two finger openings. They provide a targeted increase in pressure and an ideal pressure distribution.


Description	Item #	Universal Size
 Hand pad 8.5 x 24 cm	6844	•



Short Stretch Bandage 6846

Juzo SoftCompress Short Stretch Bandage

The short-stretch bandages exerts low resting pressure and high working pressure. They are machine washable and 100% cotton which is comfortable on the skin.


Use For	Phase 1	Item #	Carton - 10 Pieces
	Short Stretch Bandage	6846	•
	6 cm x 5 m		
	8 cm x 5 m		
	10 cm x 5 m		
	12 cm x 5 m		



Gauze Bandage 6847 / 6848

Juzo SoftCompress Gauze

The gauze consist of 60% polyamide and 40% viscose. The gauze is great for sensitive skin, joints and hard to bandage areas such as genital, fingers and toes. The Juzo Gauze is non-slip and the high elasticity of the fabric makes its application quick and easy.

Use For	Phase 1	Item #	Carton - 20 Pieces
	Gauze Bandage		
	6 cm x 4 m (finger)	6847	•
	4 cm x 4 m (toe)	6848	•

Care Instructions



Material composition




Fabric: 100 % Cotton

Foam: 100 % Polyurethane

Storage instructions

Always store Juzo SoftCompress products at room temperature and away from direct sunlight.

Washing and care instructions

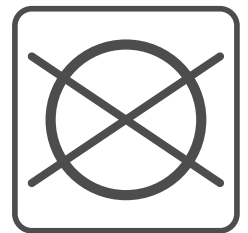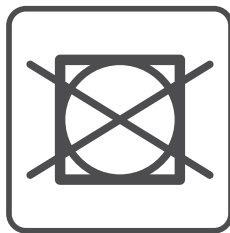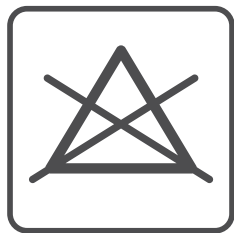

- Please wash Juzo SoftCompress products daily.
- Juzo SoftCompress products can be washed in the washing machine using a laundry detergent for delicate fabrics. Please do not use any fabric softener.
- Do not bleach Juzo SoftCompress products.
- Do not machine dry.
- You can reduce the drying time by placing Juzo SoftCompress products on a thick towel, rolling the towel up tightly, and firmly pressing out excess moisture. Do not leave Juzo SoftCompress products in the towel and do not dry on a radiator or in the sun. Dry on a flat surface.
- Do not iron Juzo SoftCompress products.
- Do not dry clean Juzo SoftCompress products.

Now Available...NEW Juzo Night

Uniform, comfortable compression for lymphedema maintenance therapy. Juzo Night is the perfect complement to flat knit compression garments worn during the day, preventing edema from rebounding during the night



Juzo  **Think Green. Live Green.**

We're committed to reducing our carbon footprint for a greener planet.

Wind
Turbine



Solar
Array



Prairie Land Grass
Landscaping



Motion Sensor
Lights



Hands-Free
Faucets & Dryers



Recycling
Centers



Juzo USA
3690 Zorn Drive
Cuyahoga Falls, OH 44223
www.juzousa.com
support@juzousa.com
1-800-222-4999



Juzo Canada Ltd.
1100 Burloak Dr. Ste. 300
Burlington, ON L7L 6B2
www.juzo.ca
support@juzo.ca
1-844-229-4999

©Juzo, Slippies, Slippie Gators, Keep Moving & Signature Collection are registered trademarks of Julius Zorn, Inc • Lycra Premium Stretch Fibers is a registered trademark of the Lycra Company. Tyvek is a registered trademark of Du Pont de Nemours and Company. • X-STATIC is a registered trademark of Noble Fiber Technologies, Inc © 2022 Julius Zorn Inc.