Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

- Short Stretch Wrap - Working pressure.
- Cross Connect - Keeps the arm wrap in place making it easy to put on and take off.
- Gap resistant - Special notches and minimal overlying straps resist gapping and bulk.
- Reversible - Worn in either beige or black

Included In This Package:

- Juzo Compression Wrap
- Juzo Hook Fasteners

Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

100 % Nylon / 100% Neoprene
Not made with natural rubber latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- If worn with a Juzo Compression Hand Wrap, the Arm Wrap should be put on first, followed by the Juzo Compression Hand Wrap placed over the Arm Wrap.

Indications

Mild to severe lymphatic edema, mild to severe lipedema

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phelgmasia coerulea dolens

Relative: Arm infection, Intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

Steps to Putting on the Arm Wrap

1. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
2. Gently fold the fasteners over onto themselves to prevent the fasteners from inadvertently attaching to the garment. Each fastener can then be unfolded before attaching.
3. Slide your arm into the wrap until the seam at the center of the wrap is positioned at the elbow.
4. Secure the bottom strap (wrist) of the wrap by pulling and fastening the strap. The strap should be just tight enough to gently hug the arm and hold the garment in place.
5. Continue working up the arm and fasten the remaining straps using enough tension for the straps to gently hug the arm.
6. Once each strap is in place, start at the bottom of the arm and tighten each strap until it feels comfortably snug against the limb.
7. Continue working up the arm and pull and tighten each strap. Make sure there are no gaps between straps. This can be best achieved by having adjacent straps slightly overlap and by inserting one strap into the notch.
8. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 4 through 7, adjusting the level of tension that is applied.

Note: If using a Juzo Compression Hand Wrap, see care and application directions for use.

JUZO COMPRESSION WRAP

Short Stretch Compression Wrap

Venda de compresión de elasticidad reducida
Bandage compressif à extensibilité courte

- High Working Pressure
- Cross Connect

REVERSIBLE

beige or black garment

Freedom in Motion

For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

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
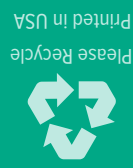
* Juzo is a registered trademark of Julius Zorn, Inc. © 2019 Julius Zorn Incorporated

1912-2019

Please Recycle
Printed in USA

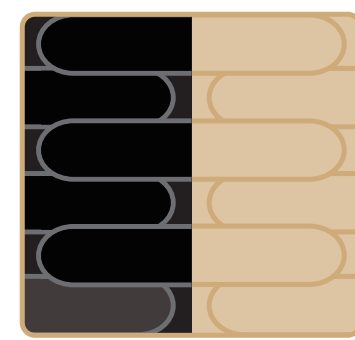GARMENT CONTENTS & CARE
See inside of insert for material contents & proper care.


REVERSIBLE
beige garment or black garment

Manufactured By
Julius Zorn, Inc • PO Box 1088
Cuyahoga Falls, OH 44223
Distributed by Juzo USA
www.juzousa.com
Distributed by Juzo Canada LTD.
www.juzo.ca

The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer. Typical resting pressure will range between 20-40 mmHg and working pressure will range between 30-60 mmHg.

C E G	CG Regular < 17" / 43cm					CG Long > 17" / 43cm	
	XS	S	M	L	XL		
27-36 cm	30-39 cm	33-44 cm	35-49 cm	40-56 cm	10 1/2"-14 1/4"	11 3/4"-15 1/2"	13 3/4"-19 1/4"
23-32 cm	24-34 cm	26-36 cm	29-42 cm	31-46 cm	9"-12 1/2"	9 1/2"-13 1/2"	10 1/4"-14 1/4"
16-19 cm	17-20 cm	18-22 cm	20-26 cm	21-29 cm	6 1/4"-7 1/2"	6 3/4"-8"	7"-8 3/4"
					8 1/4"-11 1/2"	8 3/4"-11 1/2"	8"-10 1/4"

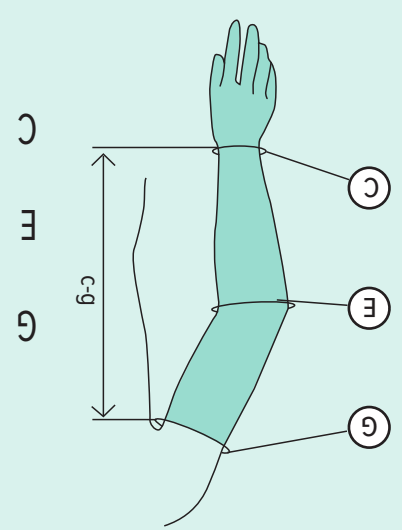Note: This size chart is not intended to replace a professional fitting by a qualified fitter.

Versatile and effective therapy.

The Juzo CompressionWrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

Short stretch for working pressure

Special notches and minimal overlying straps resist gapping and bulk

Supports movement to encourage physical activity.

Juzo Compression Wraps are made in the U.S.A.

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