- **Comfort** - Moisture & air permeable.
- **Short Stretch Wrap** - Working pressure.
- **Cross Connect** - Keeps the arm wrap in place making it easy to put on and take off.
- **Gap resistant** - Special notches and minimal overlying straps resist gapping and bulk.
- **Reversible** - Worn in either beige or black

- Juzo Reversa Compression Wrap[™]
- Juzo Hook Fasteners

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- **DO NOT USE:** bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

48 % Nylon / 46% Polyurethane / 6% Spandex
Not made with natural rubber latex.

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- **Juzo Reversa Compression Wrap® is designed to be reversible.** The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- If worn with a Juzo Reversa Compression Hand Wrap, the Arm Wrap should be put on first, followed by the Hand Wrap placed over the Arm Wrap.

Mild to severe lymphatic edema, mild to severe lipedema


Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens

Relative: Arm infection, Intolerance to compression or garment fabric, sensory disturbance to the limb.

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

1. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
2. Gently fold the fasteners over onto themselves to prevent the fasteners from inadvertently attaching to the garment. Each fastener can then be unfolded before attaching.
3. Slide your arm into the wrap until the seam at the center of the wrap is positioned at the elbow.
4. Secure the bottom strap (wrist) of the wrap by pulling and fastening the strap. The strap should be just tight enough to gently hug the arm and hold the garment in place.
5. Continue working up the arm and fasten the remaining straps using enough tension for the straps to gently hug the arm.
6. Once each strap is in place, start at the bottom of the arm and tighten each strap until it feels comfortably snug against the limb.
7. Continue working up the arm and pull and tighten each strap. Make sure there are no gaps between straps. This can be best achieved by having adjacent straps slightly overlap and by inserting one strap into the notch.
8. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 4 through 7, adjusting the level of tension that is applied.

Note: If using a Juzo Reversa Compression Hand Wrap, see care and application directions for use.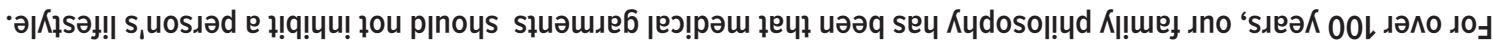

GARMENT CONTENTS & CARE

See inside of insert for material contents & proper care.

Cuyahoga Falls, OH 44223

Julius Zorn, Inc

Manufactured By



Distributed by Juzo USA www.juzousa.com

Distributed by Juzo Canada LTD. www.juzo.ca

The Juzo Reversa Compression Wrap[®] will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer. Typical resting pressure will range between 20-40 mmHg and working pressure will range between 30-60 mmHg.

A diagram of a right arm, bent at the elbow, with the forearm vertical. Three points are marked on the upper arm with circles and labels: 'C' at the shoulder, 'E' at the elbow, and 'G' at the wrist. A vertical double-headed arrow to the left of the arm spans the distance from point C to point G and is labeled 'C.g.'.

REVERSIBLE beige or black garment

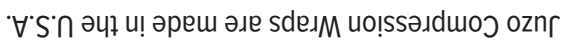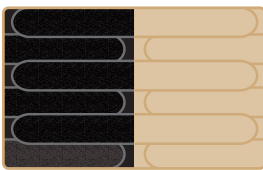The Juze Reversa Compression Wrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

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