- **Comfort** - Moisture & air permeable.
- **Working pressure** - Short stretch for a high working pressure.
- **Gap resistant** - Special notches and minimal overlying straps resist gapping and bulk.
- **EZ pull tab** - Aid for hassle free application.
- **Reversible** - Worn in either beige or black.

- Juzo Reversa Compression Wrap™
- Juzo Half Circle Hook Fasteners
- Juzo Full Foot Wrap Liner

- Remove all the half circle hook fasteners from the straps.
- Machine wash warm.
- A garment bag is strongly recommended.
- Tumble dry low.
- **DO NOT USE:** bleach, chlorine, fabric softeners, or other laundry additives.
- Store garments in a dry place at room temperature.

48% Nylon / 46% Polyurethane / 6% Spandex
Not made with natural rubber latex.

- Juzo Reversa Compression Wrap™ is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- Juzo Full Foot Liner option provides a sanitary layer between the garment and the skin and/or any wound dressing.

The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

1. The Juzo Reversa Compression Wrap™ comes with a full foot liner.
 - If using the Juzo Wrap Liner, slide the liner over the foot and heel, pulling up the leg and over the calf. Position the top of the liner just below the knee, making sure the liner is smooth from top to bottom.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. Gently fold the fasteners over onto themselves to prevent the fasteners from inadvertently attaching to the garment. Each fastener can then be unfolded before attaching.
4. While in a resting position, i.e. sitting down, position the wrap on the leg just below the knee. (2 fingers below the knee). The seam should run down the center of the backside of the leg.
5. Secure the top strap of the wrap by counter pulling the top strap and the top EZ Pull at the same time. The strap should be just tight enough to gently hug the leg and hold the garment in place.
6. Continue working down the leg and fasten the remaining straps using enough tension for the straps to gently hug the leg. We recommend simultaneously counter pulling the two adjacent straps.
7. Once each strap is in place, start at the bottom of the leg and in unison pull the bottom strap and EZ Pull in opposite directions.
8. Continue working up the leg and counter pull the second and third straps. Make sure there are no gaps between straps. This can be best achieved by having adjacent straps slightly overlap and by inserting one strap into the notch.
9. Pull the top strap and the EZ Pull until it is comfortably tightened.
10. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 6 through 8, adjusting the level of tension that is applied.

Note: If you are using a Juzo Compression Wrap Foot Segment, see care and application directions for use.

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