Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

- Short Stretch Wrap - Working pressure
- Reversible - Worn in either beige or black

Included In This Package:

- Juzo Compression Wrap
- Juzo Hook Fasteners

Care and Maintenance

- Remove both the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

100% Nylon / 100% Neoprene - Not made using natural rubber or latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- If worn with a Juzo Compression Arm Wrap, the Arm Wrap should be put on first, followed by the Juzo Compression Hand Wrap placed over the Arm Wrap.

Indications

Mild to severe Lymphatic edema, mild to severe Lipedema

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phelgmasia coerulea dolens.

Relative: Hand infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

Steps to Putting on the Hand Wrap

1. The Juzo Compression Hand Wrap is designed to be reversible. However, for more optimal results, Juzo recommends putting the garment on as described below.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. If using with a Juzo Compression Arm Wrap, the Arm Wrap should be donned first, followed by the Juzo Hand Wrap placed over the Arm Wrap.
4. To don the hand wrap, slide thumb in the thumb stub so that the short side of the wrap is on the palm side. Wrap the garment around the the hand and bring the thumb through the thumb hole. Attach the fasteners, beginning with the wrist strap, until the hand wrap fits comfortably snug.
5. Pull the straps at the wrist and on the back of the hand until they are comfortably snug and fasten the straps to the body of the garment.

Note: If using a Juzo Compression Arm Wrap, see care and application directions for use.

JUZO COMPRESSION WRAP

Short Stretch Compression Wrap

Venda de compresión de elasticidad reducida
Bandage compressif à extensibilité courte

- Working Pressure

REVERSIBLE

beige or black garment

Freedom in Motion

For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

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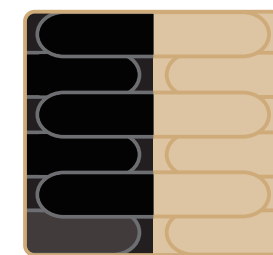
Juzo is a registered trademark of Julius Zorn, Inc. © 2019 Julius Zorn Incorporated

INHANDWRAP-1218

Printed in USA
Please Recycle

GARMENT CONTENTS & CARE
See inside of insert for material contents & proper care.

REVERSIBLE
beige garment or black garment

Manufactured By
Julius Zorn, Inc. • PO Box 1088
Cuyahoga Falls, OH 44223
Distributed by Juzo USA
www.juzousa.com
Distributed by Juzo Canada LTD.
www.juzo.ca

The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

Size Chart For Juzo Compression Hand Wrap

Note: This size chart is not intended to replace a professional fitting by a qualified fitter.

	XS	S	M	L	XL
A	15.5 - 17.5cm 6 1/4" - 7 3/4"	17.5 - 19.5cm 7 3/4" - 8 3/4"	19.5 - 22cm 8 3/4" - 9 1/2"	22 - 24.5cm 9 1/2" - 11"	24.5 - 28cm 9 1/2" - 11"
C	13-17cm 5" - 6 1/2"	15-19cm 5 3/4" - 7 1/2"	17-21.5cm 7 1/2" - 8 1/2"	20-24cm 8" - 9 1/4"	21-25cm 8 1/4" - 9 3/4"

Fit using point A

The Juzo CompressionWrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

Versatile and effective therapy.

Supports movement to encourage physical activity.

Juzo Compression Wraps are made in the U.S.A.

Short stretch for working pressure

Special notches and minimal overlying straps resist gapping and bulk

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