Manufactured with care and detail, your Reversa Juzo Compression Wrap™ has several features that provide therapy, comfort, versatility and superior fit, including:

- **Comfort** - Moisture & air permeable.
- **Short Stretch Wrap** - Working pressure
- **Reversible** - Worn in either beige or black

Included In This Package:

- Juzo Reversa Compression Wrap™
- Juzo Hook Fasteners

Care and Maintenance

- Remove both the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- **DO NOT USE:** bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

48% Nylon / 46% Polyurethane / 6% Spandex - Not made using natural rubber or latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Reversa Compression Wrap™ is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- If worn with a Juzo Reversa Compression Arm Wrap, the Arm Wrap should be put on first, followed by the Juzo Hand Wrap placed over the Arm Wrap.

Indications

Mild to severe Lymphatic edema, mild to severe Lipedema

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens.

Relative: Hand infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

Steps to Putting on the Hand Wrap

1. The Juzo Reversa Compression Hand Wrap is designed to be reversible. However, for more optimal results, Juzo recommends putting the garment on as described below.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. If using with a Juzo Reversa Compression Arm Wrap, the Arm Wrap should be donned first, followed by the Juzo Hand Wrap placed over the Arm Wrap.
4. To don the hand wrap, slide thumb in the thumb stub so that the short side of the wrap is on the palm side. Wrap the garment around the the hand and bring the thumb through the thumb hole. Attach the fasteners, beginning with the wrist strap, until the hand wrap fits comfortably snug.
5. Pull the straps at the wrist and on the back of the hand until they are comfortably snug and fasten the straps to the body of the garment.

Note: If using a Juzo Reversa Compression Arm Wrap, see care and application directions for use.

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