Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

- Working pressure - Short stretch for a high working pressure.
- EZ pull tab - Aid for hassle free application.
- Reversible - Worn in either beige or black

#### Included In This Package:

- Juzo Compression Wrap
- Juzo Hook Fasteners

#### Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

#### Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

#### Fabric

100 % Nylon / 100% Neoprene  
Not made with natural rubber latex.

#### Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.

#### Indications

Primary & secondary varices, chronic venous insufficiency, post-thrombotic syndrome, venous stasis ulcers, pre and post-vein procedure and lymphatic edema.

#### Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens.

Relative: Leg infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

#### Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage /injuries which have been caused by improper handling or misuse.

#### Steps to Putting on the Knee Wrap

1. Prior to applying the knee wrap you should have the calf wrap segment on your leg.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. While sitting down, extend your leg and position the wrap on your knee so that the horizontal seam aligns with the bend in the back of your knee.
4. Secure the top strap of the wrap by counter pulling the top strap and the top EZ Pull on the opposite side of the garment at the same time. The strap should be just tight enough to gently hug the leg and hold the garment in place.
5. After the top strap is in place, pull the bottom knee strap and in- unison pull the EZ tab on the opposite side of the garment.
6. Once the knee wrap is positioned on your leg again pull the top strap along with the EZ Pull until it is comfortably snug. When pulling the strap, slightly pull it downward toward the knee center.
7. Repeat this step with the lower strap, only this time pull the lower strap upwards toward the knee center. The strap should be tightened until it is comfortably snug.
8. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 6 through 7, adjusting the level of tension that is applied.

Note: See care and application directions for use of Juzo Compression Wrap Thigh, Calf & Foot Segment.  
Extra fasteners are provided inside packaging.

# JUZO COMPRESSION WRAP

Short Stretch Compression Wrap

Venda de compresión de elasticidad reducida  
Bandage compressif à extensibilité courte

- High Working Pressure
- EZ Pull Tabs

**REVERSIBLE**  
beige or black garment

Freedom in Motion


For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

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Juzo is a registered trademark of Julius Zorn, Inc. © 2019 Julius Zorn Incorporated. INKNEWRAP 1218 Printed in USA Please Recycle

GARMENT CONTENTS & CARE  
See inside of insert for material contents & proper care.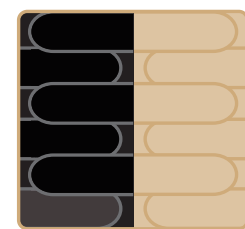

Manufactured By  
Julius Zorn, Inc. • PO Box 1088  
Cuyahoga Falls, OH 44223  
Distributed by Juzo USA  
www.juzousa.com  
Distributed by Juzo Canada LTD.  
www.juzo.ca

The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

Size Chart For Juzo Compression Knee Wrap

Note: This size chart is not intended to replace a professional fitting by a qualified fitter.

Patella	E	Mid	30-36 cm	11 3/4"-14 1/4"	36-42 cm	14 1/4"-16 1/2"	42-48 cm	16 1/2"-19"	48-55 cm	19"-21 3/4"	55-62 cm	21 3/4"-24 1/4"	62-70 cm	21 3/4"-27 1/2"
			S		M		L		XL		2XL		3XL	

PCSZ-01-55

- RESISTANT: Special stretch for a high working pressure
- USA: Special notches and minimal overlying straps resist gapping and bulk
- EZ PULL TABS: Pull tab aid for hassle-free application
- MADE IN THE USA: Juzo Compression Wraps are made in the U.S.A.

The Juzo CompressionWrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

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