Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Reversa Compression Wrap™ has several features that provide therapy, comfort, versatility and superior fit, including:

- **Comfort** - Moisture & air permeable.
- **Working pressure** - Short stretch for a high working pressure.
- **Gap resistant** - Special notches and minimal overlying straps resist gapping and bulk.
- **EZ pull tab** - Aid for hassle free application.
- **Reversible** - Worn in either beige or black.

Included In This Package:

- Juzo Reversa Compression Wrap™ with Slip On Feature
- Juzo Half Circle Hook Fasteners
- Juzo Full Foot Wrap Liner

Care and Maintenance

- Remove all the half circle hook fasteners from the straps.
- Machine wash warm.
- A garment bag is strongly recommended.
- Tumble dry low.
- **DO NOT USE:** bleach, chlorine, fabric softeners, or other laundry additives.
- Store garments in a dry place at room temperature.

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

48% Nylon, 46% Polyurethane, 6% Spandex, Inner Shell - 100% Neoprene • Not made with natural rubber latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Reversa Compression Wrap™ with Slip On Feature is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- Juzo Full Foot Liner option provides a sanitary layer between the garment and the skin and/or any wound dressing.

Indications

Primary & secondary varices, chronic venous insufficiency, post-thrombotic syndrome, venous stasis ulcers, pre and post-vein procedure and lymphatic edema.

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phelgmasia coerulea dolens.

Relative: Leg infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. Juzo Silver Garments should be removed prior to MRI Scans & Cardiac Defibrillations.

The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

Steps to Putting on the Calf Wrap with Slip On Feature

1. The Juzo Reversa Compression Wrap™ with Slip On Feature comes with an optional Juzo Wrap Liner.
 - If using the Juzo Wrap Liner, slide the liner over the foot and heel, pulling up the leg and over the calf. Position the top of the liner just below the knee, making sure the liner is smooth from top to bottom.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. Gently fold the fasteners over onto themselves to prevent the fasteners from inadvertently attaching to the garment. Each fastener can then be unfolded before attaching.
4. While in a resting position, i.e. sitting down, slide your foot through the slip on liner, pulling the wrap up the leg and position the wrap just below the knee. (2 fingers below the knee). The seam should run down the center of the backside of the leg.
5. Secure the top strap of the wrap by counter pulling the top strap and the top EZ Pull at the same time. The strap should be just tight enough to gently hug the leg and hold the garment in place.
6. Continue working down the leg and fasten the remaining straps using enough tension for the straps to gently hug the leg. We recommend simultaneously counter pulling the two adjacent straps.
7. Once each strap is in place, start at the bottom of the leg and in unison pull the bottom strap and EZ Pull in opposite directions
8. Continue working up the leg and counter pull the second and third straps. Make sure there are no gaps between straps. This can be best achieved by having adjacent straps slightly overlap and by inserting one strap into the notch.
9. Pull the top strap and the EZ Pull until it is comfortably tightened
10. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 6 through 8, adjusting the level of tension that is applied.

Note: If you are using a Juzo Reversa Compression Wrap™ Foot Segment, see care and application directions for use.

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Printed in USA

Product Information

For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

© 2025 Julius Zorn, Inc

Julius Zorn, Inc
3690 Zorn Drive
Cuyahoga Falls, OH 44223

Distributed by Juzo Canada LTD.
www.juzo.ca

GARMENT CONTENTS & CARE

See inside of insert for material contents & proper care.

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® Juzo is a registered trademark of Julius Zorn, Inc

© 2025 Julius Zorn Incorporated

A

B

C

D

length AD

Max Calf

Calf

Calf

Ankle

35-41 cm

29-35 cm

21-24 cm

18-21 cm

13 3/4"-16 1/2"

13 1/4"-13 3/4"

10 3/4"-12 1/4"

7"-8 1/4"

40-46 cm

34-40 cm

27-31 cm

24-27 cm

15 3/4"-18"

14 1/2"-17 1/4"

12 1/4"-13 3/4"

9 1/2"-10 3/4"

44-51 cm

37-44 cm

27-31 cm

24-27 cm

17 1/4"-20"

16 1/4"-19"

12 1/4"-12 1/4"

9 1/2"-10 3/4"

48-55 cm

41-48 cm

27-31 cm

24-27 cm

19"-21 3/4"

16 1/4"-19"

12 1/4"-12 1/4"

9 1/2"-10 3/4"

53-60 cm

46-53 cm

31-35 cm

24-27 cm

20 3/4"-23 1/2"

18"-20 3/4"

12 1/4"-13 3/4"

9 1/2"-10 3/4"

60-70 cm

50-60 cm

36-47 cm

24-27 cm

23 1/2"-27 1/2"

19 3/4"-23 1/2"

14"-18 1/2"

11 1/2"-13 3/4"

AD Regular < 40cm / 15 3/4"

AD Long > 40cm / 15 3/4"

PCSZ-01-51

Note: This size chart is not intended to replace a professional fitting by a qualified fitter.

REVERSIBLE

beige or black garment

Short stretch for working pressure

Moisture and air permeable for increased comfort.

Special notches and minimal overlying straps resist gapping and bulk

Julius Zorn Compression Wraps are made in the U.S.A.

The Juzo Reversa Compression Wrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

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- Juzo Slip On Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- Juzo optional full foot liner provides a sanitary layer between the garment and the skin and/or any wound dressing.

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