

# Juzo Prosthetic Shrinker

- Therapeutic compression
- Soft, breathable knit



Freedom in Motion




Manufactured by  
Juliuz Zorn, Inc.  
PO Box 1088  
Cuyahoga Falls, OH 44223  
Juzo USA  
www.juzousa.com  
Juzo Canada LTD.  
www.juzo.ca  
© 2019 Juliuz Zorn Incorporated  
Juzo and Keep Moving are registered trademarks of Juliuz Zorn, Inc.  
INSTUMP-1019

## 20-30 and 30-40 mmHg

### Size Chart for Juzo Below Knee Shrinker

BK Lengths: Long - 18" / 46 cm Medium - 15" / 38 cm Short - 12" / 30 cm

	0	I	II	III	IV	V	VI
F	38 cm 15 1/4"	41 cm 16 1/4"	44 cm 17 1/4"	47 cm 18 1/2"	50 cm 19 3/4"	53 cm 20 3/4"	56 cm 22"
E	28 cm 11 1/4"	31 cm 12 1/4"	34 cm 13 1/2"	37 cm 14 1/2"	40 cm 15 3/4"	43 cm 17"	46 cm 18"
C	26 cm 10 1/2"	29 cm 11 1/2"	32 cm 12 1/2"	35 cm 13 3/4"	38 cm 15"	41 cm 16 1/4"	44 cm 17 1/2"

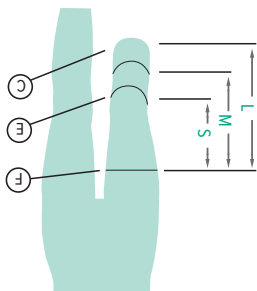


## 20-30 and 30-40 mmHg

### Size Chart for Juzo Above Knee Shrinker

AK Lengths: Extra Long - 14" / 35 cm Long - 12" / 30 cm Medium - 10" / 25 cm Short - 8" / 20 cm

	I	II	III	IV	V	VI
G	44 cm 17 1/4"	48 cm 19"	52 cm 20 1/2"	56 cm 22"	60 cm 23 3/4"	64 cm 25 1/4"
F	41 cm 16 1/4"	44 cm 17 1/4"	47 cm 18 1/2"	50 cm 19 3/4"	53 cm 20 3/4"	56 cm 22"
E	31 cm 12 1/4"	34 cm 13 1/2"	37 cm 14 1/2"	40 cm 15 3/4"	43 cm 17"	46 cm 18"

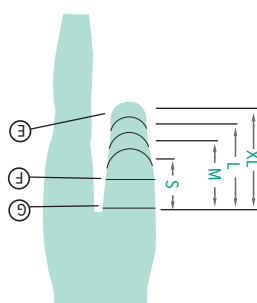

PCSZ-01-08

PCSZ-01-08

**Freedom to Make the Most of Every Day**

Juzo prosthetic shrinkers are designed to improve your quality of life, not inhibit it. Family-owned since 1912, Juzo has led the compression therapy garment industry by creating high-quality products that offer you the freedom to maintain a more active lifestyle. Thank you for choosing Juzo.

**Care and Maintenance**

To maintain your Juzo shrinker, we recommend the following:

- Hand or machine wash warm daily to maximize therapeutic effectiveness and prolong the life of your Juzo garment.
- When machine washing and drying, use the lowest permanent press settings. A garment bag is strongly recommended and available in the Juzo Compression Therapy Careaccessory package.
- Wash dark colors separately from other clothing.
- Use Juzo washing detergent or another mild soap.
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives. These may damage your garment.
- Please refer to the washing symbols on the label sewn into your Juzo shrinker.
- Store your Prosthetic Shrinker in a dry place at room temperature.

**Return Policy**

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase.

**Fabric**

Details are provided on label sewn inside your prosthetic shrinker.

**Instructions for Use**

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. As therapy progresses or new symptoms develop, it may be necessary to take new measurements for a new prosthetic shrinker.

1. Turn the shrinker inside out, except for its end part.
2. Take the shrinker, folded over itself, in both hands and slip it over the residual limb.
3. Turn the shrinker back and pull it evenly over the residual limb.
4. Now stroke the whole shrinker gradually upwards, smoothing it to your leg with the palm of your hand.

**Important:** Never pull at the upper edge of the prosthetic shrinker

**Indications**

Age or trauma related amputation, prevention of edemas after removal of prosthesis improving circulation in the residual limb.

The compression class must be determined by your physician. Regular medical visits will help to ensure that your therapy is as successful as possible.

**Contraindications\***

Open wounds, weeping skin areas, arterial insufficiency, stasis dermatitis, acute thrombophlebitis, non reduced edemas, non compensated cardiac insufficiency.

Julius Zorn Inc assumes no liability as a result of any contraindicated use of this product.

**Cautions**

It is recommended that Juzo Silver Garments be removed prior to cardiac defibrillation and MRI scans. No side effects are currently known. However if a negative result should occur (i.e. skin irritations) from the use of our product, please consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

**Always use a Juzo donning glove when applying compression garments.**