- **Comfort** - Moisture & air permeable.
- **Working pressure** - Short stretch for a high working pressure.
- **Gap resistant** - Special notches and minimal overlying straps resist gapping and bulk.
- **EZ pull tab** - Aid for hassle free application.
- **Reversible** - Worn in either beige or black.

- Juzo Reversa Compression Wrap™
- Juzo Hook Fasteners

- Remove all the half circle hook fasteners from the straps.
- Machine wash warm.
- A garment bag is strongly recommended.
- Tumble dry low.
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives.
- Store garments in a dry place at room temperature.

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

48% Nylon / 46% Polyurethane / 6% Spandex • Not made with natural rubber latex.

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Reversa Compression Wrap™ is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposite side of the color to be worn.

Primary & secondary varices, chronic venous insufficiency, post-thrombotic syndrome, venous stasis ulcers, pre and post-vein procedure and lymphatic edema.





Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens.

Relative: Leg infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. Juzo Silver Garments should be removed prior to MRI Scans & Cardiac Defibrillations.


The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

Prior to applying the thigh wrap you should have the knee wrap segment in place and on your leg. The thigh wrap should slightly overlap the top of the knee wrap.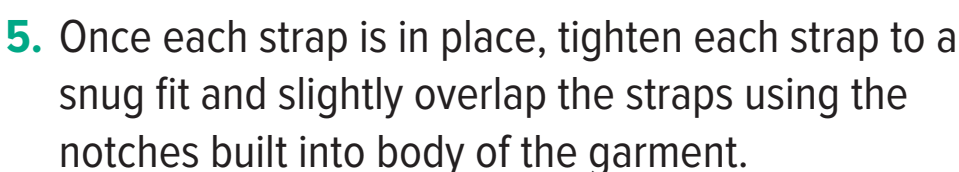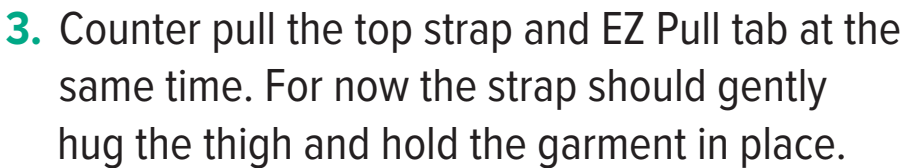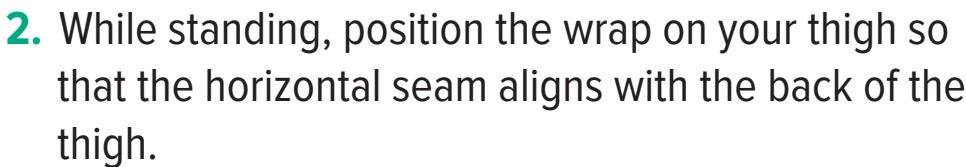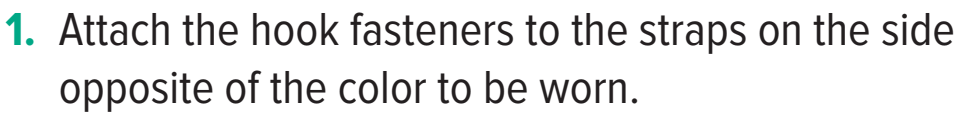

If you feel any pain or discomfort or the wrap feels too loose, then repeat steps adjusting the level of tension that is applied.

**Note:** See care and application directions for use of Juzo Compression Wrap Foot, Calf & Knee Segment. Extra fasteners are provided inside packaging.

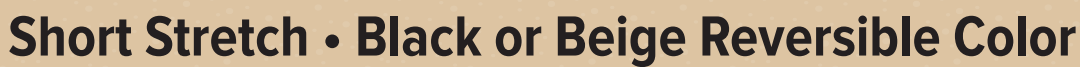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© 2025 Julius Zorn Incorporated. JuZo is a registered trademark of Julius Zorn, Inc.

For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

Distributed by Juzo USA [www.juzousa.com](http://www.juzousa.com)  
Distributed by Juzo Canada LTD. [www.juzo.ca](http://www.juzo.ca)

Manufactured By  
Julius Zorn, Inc  
Cuyahoga Falls, OH 44223

**GARMENT CONTENTS & CARE**  
See inside of insert for material  
contents & proper care.

The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

Special notches and minimal overlying straps resist gapping and bulk

The Juzo Reversa Compression Wrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

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