

JUZO COMPRESSION WRAP

Short Stretch Compression Wrap

- Hip Attachment

REVERSIBLE

beige or black garment

Freedom in Motion



For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.

Venda de compresion de elasticidad reducida para piernas saludables y siempre en movimiento.

Bandage compressif à extensibilité courte pour aider à améliorer la circulation et garder

les jambes en bonne santé et en mouvement.


© 2019 Juzo Zorn, Inc. Juzo is a registered trademark of Julius Zorn, Inc.

INWRHIPATTACH-019

Please Recycle
Printed in USA



GARMENT CONTENTS & CARE
See inside of insert for material
contents & proper care.

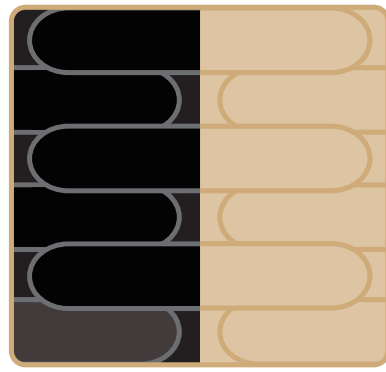

REVERSIBLE
beige garment or black garment

Manufactured By
Julius Zorn, Inc • PO Box 1088
Cuyahoga Falls, OH 44223
Distributed by Juzo USA
www.juzousa.com
Distributed by Juzo Canada LTD.
www.juzo.ca

The Juzo Compression Wrap Hip Attachment is one size fits all.
The belt length on the Juzo Compression Wrap Hip Attachment is 140cm (55 inches) long. The belt can be cut shorter to the appropriate length if needed.
The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

Versatile and effective therapy.

Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit.

Included In This Package:

- Juzo Hip Attachment

Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

100 % Nylon / 100% Neoprene

Not made with natural rubber latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guide lines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

Use fasteners and attach to the waist belt. If desired the belt can be trimmed to an appropriate length that fits comfortably around the waist. If the belt is trimmed, re-position fasteners to the belt so you can secure around your waist. Once the belt is secured, add the 6 inch fastener to the lowest part of the hip attachment and attach to the thigh compression wrap.

- Juzo Hip Attachment is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage/injuries which have been caused by improper handling or misuse.

Note: See care and application directions for use of Juzo Compression Wrap Thigh, Knee, Calf & Foot Segment. Extra fasteners are provided inside packaging.