

IMPROVED
MATERIAL!



Juzo

REVERSA

COMPRESSION WRAP

Short Stretch • Black or Beige Reversible Color

Venda de compresión de elasticidad reducida

Bandage compressif à extensibilité courte

- Moisture & Air Permeable
- High Working Pressure
- High Containment
- EZ Pull Tabs

HIP ATTACHMENT



Please Recycle
Printed in USA



Product Information



Venda de compresión de elasticidad reducida para piernas saludables y siempre en movimiento.

Bandage compressif à extensibilité courte pour aider à améliorer la circulation et garder

les jambes en bonne santé et en mouvement.


INHIPATTACH-0524

Versatile and effective therapy.

The Juzo Reversa Compression Wrap™ Hip Attachment is one size fits all.

The belt length on the Juzo Reversa Compression Wrap™ Hip Attachment is 140cm (55 inches) long. The belt can be cut shorter to the appropriate length if needed.

The Juzo Reversa Compression Wrap™ will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

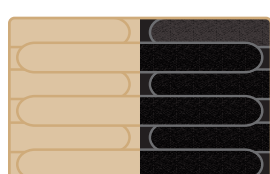

REVERSIBLE
beige or black garment

GARMENT CONTENTS & CARE
See inside of insert for material
contents & proper care.

Manufactured By
Julius Zorn, Inc
3690 Zorn Drive
Cuyahoga Falls, OH 44223

Distributed by Juzo Canada LTD.
www.juzo.ca

For over 100 years, our family philosophy has been that medical garments
should not inhibit a person's lifestyle.



Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

Included In This Package:

- Juzo Hip Attachment

Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

48% Nylon / 46% Polyurethane / 6% Spandex

Not made with natural rubber latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guide lines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

Use fasteners and attach to the waist belt. If desired the belt can be trimmed to an appropriate length that fits comfortably around the waist. If the belt is trimmed, re-position fasteners to the belt so you can secure around your waist. Once the belt is secured, add the 6 inch fastener to the lowes part of the hip attachment and attach to the thigh compression wrap.

- Juzo Hip Attachment is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage/injuries which have been caused by improper handling or misuse.

Note: See care and application directions for use of Juzo Compression Wrap Thigh, Knee, Calf & Foot Segment. Extra fasteners are provided inside packaging.