Medical Compression Stockings



Medical compression garments from Juzo guarantee therapeutic graduated compression. The pressure decreases continuously from the ankle or wrist section upward towards the distal end. We can provide a wide variety of garments, from below-knee stockings to pantyhose, from gloves to face mask, depending on the therapeutic application.

RTW - Standard / MTM - Custom

RIW - Standard / MIM - Custom									
Mild Compression	RTW - Circular	15-20 mmHg							
	 minor obstructive disorders accompanied by feelings of tiredness or heaviness in the le isolated varicose vein formation preventative measure during pregnancy therapeutic treatment of scars 								
Moderate Compression	RTW - Circular	20-30 mmHg	MTM - Flat / Circular Knit	18-21 mmHg					
	after varicosefor the long-tpropensity fo	during pregnancy e vein sclerosis or operation erm treatment of healed leader edema formation exists							
Firm Compression	RTW - Circular	reatment of scars 30-40 mmHg	MTM - Flat / Circular Knit	23-32 mmHg					
	after healingwhen a sevenchronic veno	of venous ulcers re swelling tendency exists us insufficiency ensity for edema formation	S						
Strong Compression	RTW - Circular	40-50 mmHg	MTM - Flat / Circular Knit	34-46 mmHg					
		nedema nic venous insufficiency t of active venous uclers							
Extra Strong Compression	Circular N/A		MTM - Flat Knit	50+ mmHg					
	severe post tsevere lymph	hrombotic conditions							

· elephantiasis



Prescription Recommendation for Venous Disease

Indications	Juzo Naturally Sheer	Juzo Power Collection	Juzo Basic	Juzo Soft	Juzo Dynamic	Juzo Move	Juzo Silver	Juzo Compression Wrap
Minor varices								
Beginning varices during pregnancy								
Prophylaxis of venous disorders	•	•	•	•		•		•
for mobile patients								
CVI stage 1	•	•	•	•	•	•		•
After thrombophlebitis	•	•	•	•	•	•	•	•
Primary varices								
Secondary varices	•	•	•	•	•	•		•
Distinctive varices during pregnancy								
Vein stripping								
Sclerotherapy								
After removal of leg veins (e.g. bypass	•	·	•	•	· ·	•		·
surgery heart)								
CVI stage 2			•	•	•	•	•	•
Atrophie blanche					•		•	•
Phlebothrombosis					•		•	•
After healing of heavy venous ulcers					•		•	•
Thrombophlebitis			•	•	•	•	•	•
Severe secondary varices					•		•	•
CVI stage 3					•		•	•
Post-thrombotic syndrome					•		•	•

Classification regarding CEAP (Clinical Etiology Anatomy Pathophysiology)

Clinical classification	Juzo Naturally Sheer	Juzo Power Comfort, Cotton	Juzo Power Lite, Vibe	Juzo Power Wool	Juzo Basic	Juzo Soft	Juzo Dynamic	Juzo Move	Juzo Silver	Juzo Ulcer Pro	Juzo Compression Wrap
C ₁ Spider veins, reticular varices	•	•	•	•	•	•		•			
C ₂ Varices	•	•	•	•	•	•	•	•	•		•
C ₃ Edema		•	•	•	•	•	•	•	•		•
C _{4a} Pigmentation, purpura, eczema									•		•
C _{4b} Hypodermitis, lipodermatosclerosis, atrophie blanche							•		•		•
C ₅ Healed ulcus cruris venosum					•	•	•	•	•		•
C ₆ Active ulcus cruris venosum							•		•	•	•

Etiologic classification

Ec congenital

Ep primary

Es secondary (post-thrombotic)

Anatomic classification

s superficial veins

p perforator veins

d deep veins

Pathophysiologic classification

Pr reflux

Po obstruction

Pr,o reflux and obstruction



Prescription Recommendation for Lymphedema & Lipedema

			DECONGESTION PHASE	MAINTENANCE PHASE									
Indication	Stage	Symptom	Juzo Compression Wrap	Juzo Night	Juzo Move	Juzo Basic	Juzo Dynamic/ Dynamic Silver	Juzo Soft	Juzo Sensation	Juzo Expert / Expert Cotton	Juzo Expert Silver	Juzo Strong	Juzo Strong Silver
Lymphedema	1	Spontaneously reversible, soft swelling, pressing on the skin leaves a pit, no skin changes		•	•	•		•		•			
	2	Spontaneously irreversible, onset of fibrosis, very little pitting, sometimes skin changes	•							•		•	•
	3	Advanced form, elephantiasis, fibrosis, chronic skin changes	•									•	
Lipedema	1	Fine-nodule skin surface (orange peel skin), fatty swelling in the hip area			•	•	•	•	•	•			
	2	Coarse-nodule, uneven skin surface with large depressions								•		•	•
	3	Large, deformed skin flaps and lobules, with the tissue being firmer and harder	•							•	•		•

Scan to download the Juzo Digital Catalog & Decision Tree App



Juzo USA P.O. Box 1088 Cuyahoga Falls, OH 44223 www.juzousa.com support@juzousa.com Juzo Canada Ltd. 1100 Burloak Dr. Ste. 300 Burlington, ON L7L 6B2 www.juzo.ca support@juzo.ca