Please take a few minutes to register your product online to enable us to provide you with the best customer service possible.

Benefits of Registration:

Notification

Allows us to contact you with important product notifications and updates.

Confirmation of Ownership

Provides us with a record

Improved Product Development

Help us continue to design quality products with your feedback.

Register your Juzo product online at:
www.juzoproducts.com

Juzo USA
P.O. Box 1088
Cuyahoga Falls, OH 44223
1 888-255-1300
www.juzousa.com
support@juzousa.com

Juzo Canada Ltd.
1100 Burloak Dr. Ste. 300
Burlington, ON L7L 6B2
1 888-255-1300
www.juzo.ca
support@juzo.ca

Freedom to Make the Most of Every Day

Riding a bike. Working in the garden. Playing with kids. Taking a walk. Going to work. These are the keep moving benefits of Juzo compression garments, designed to improve your quality of life, not inhibit it. Family-owned since 1912, Juzo has led the compression therapy garment industry by creating high-quality products that offer you the freedom to maintain a more active lifestyle. Advancements like our FiberSoft technology, seasonal colors, attractive styles and fun print designs ensure you move in comfort without compromising fashion. Thank you for choosing Juzo.

Care and Maintenance

To maintain your Juzo arm sleeve, we recommend the following:

- Hand or machine wash warm daily to maximize therapeutic effectiveness and prolong the life of your Juzo garment.
- When machine washing and drying, use the lowest permanent press settings. A garment bag is strongly recommended and available in the Juzo Compression Therapy Care accessory package.
- Wash dark colors separately from other clothing.
- Use Juzo washing detergent or another mild soap.
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives. These may damage your garment.
- Please refer to the washing symbols on the label sewn into your Juzo arm sleeve.
- Store your arm sleeve in a dry place at room temperature.

Return Policy

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase.

Fabric

Details are provided on label sewn inside your compression arm sleeve.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. As therapy progresses or new symptoms develop, it may be necessary to take new measurements for a new compression arm sleeve.

It is also recommended that you put your compression arm sleeve on first thing in the morning every day. Rough fingernails, jagged rings etc. can damage fabric; use a donning glove to protect the garment.

Indications

Lymphedema and edema management, venous congestion, follow up treatment of burns and scars, post-traumatic edema.

Contraindications*

Arterial insufficiency.

Julius Zorn Inc assumes no liability as a result of any contraindicated use of this product.

Cautions

It is recommended that Juzo Silver Garments be removed prior to cardiac defibrillation and MRI scans. No side effects are currently known. However if a negative result should occur (i.e. skin irritations) from the use of our product, please consult with a medical professional. The manufacturer is not liable for damage / injuries which have been caused by improper handling or misuse.

© Juzo, Slippie Gator, Keep Moving, Fiber Soft & Signature Collection are registered trademarks of Julius Zorn, Inc. Lycra is a registered trademark of Invista Inc. Tyvek is a registered trademark of Dupont.
© 2021 Julius Zorn, Inc.

Care & Application
Compression Arm Sleeves

Useful tips to help with donning your compression arm sleeve:


1 Fold the top half of the sleeve down over the bottom half of the sleeve.

2 Using the folded edge, gently ease the sleeve over the hand and up the arm until the folded edge reaches the elbow. For sleeves with a shoulder strap, Make sure the shoulder attachment remains on the outer part of the arm.

3 Using Juzo donning gloves, take the end of the top half of the sleeve and slide it slowly up the upper part of the arm towards the shoulder.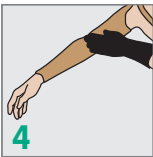

4 Use the palm of your hand to equally distribute the knit and guide the sleeve into place. The knit should look even and fit smoothly over the length of the arm.

5 For sleeves with a shoulder strap, position one strap across the chest and the other around the back. Fasten the two ends together. Note: Should the strap be too tight or too loose, the length of the strap can be adjusted using the adjustment clip.

Always use Juzo donning gloves when applying compression garments.

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