



## Compression Segments Head

### General instructions

- Please bear in mind that the cutouts need to be kept as small as possible so as not to reduce the compression effect or therapy results. Too large cutouts could result in "window oedemas".
- In those styles with a closed forehead section, the entire back of the head is also closed.  
In the case of the variant with an open forehead section, the back of the head is also open up to the height of the forehead.

## Circumference measurements



### Circumference "cA"

Lower neck circumference  
parallel to the circumference  
"cB"



### Circumference "cB"

Neck circumference directly  
below the chin and over the  
first cervical vertebra at the  
back of the head



### Circumference "cC"

Over the tip of the chin and  
over the first cervical vertebra  
at the back of the head



### Circumference "cD"

Directly below the lower lip  
and over the first cervical ver-  
tebra at the back of the head



### Circumference "cE"

Directly above the eyebrows



### Circumference "cE'"

Parallel to the circumference "cE"

# Length measurements



**Length "LAB"**  
Width of the neck part: between "cB" and "cA", measuring at the front of the neck and following the body. Ensure that the distance is not so small that the fabric could roll up.



**Length "LBC"**  
Directly below the chin to the tip of the chin, measuring at the front of the neck and following the body



**Length "LCD"**  
From the tip of the chin to below the lower lip, measuring at the front and following the body



**Length "LD'D"**  
Headband from point "D" over the largest area of the head to the point "D" on the opposite side



**Length "LEE"**  
Width of the headband: between "cE" and "cE"



**Length "LDE"**  
Position of the headband: between "cD" and "cE"

# Additional measurements for face masks

**Width measurements** Please measure horizontally.



**Width "K"**  
Mouth cutout, from one corner of the mouth to the other



**Width "M"**  
Nose cutout, horizontally from the medial side of each nasal wing



**Width "M1"**  
Knitted-in nose: from nasal wing to nasal wing over the tip of the nose measuring loosely



**Width "N"**  
Eye cutout, from the corner of one eye to the corner of the other



**Width "P"**  
Distance between the eyes



**Width "Ear"**  
Ear cutout

**Length measurements** Please measure vertically.



**Length "M2"**  
Root of the nose to the tip of the nose



**Length "S"**  
Upper lip to the nose



**Length "T"**  
Below the lower lip to the eye



**Length "U"**  
Height of the eye cutout



**Height "Ear"**  
Ear cutout